


## Disinfection protocol

After cleaning allow the disinfectant to evaporate between sessions.  
Use the device when all surfaces of the device are dry.

### Cleaning materials

medical disinfectant (with  
max. 70% isopropyl alcohol)

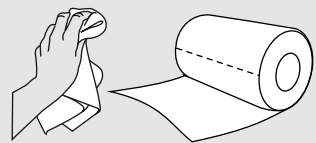


Soft non-scratching disposable  
cleaning towel




Do **not** use the provided  
microfiber cloth.



### Cleaning the lightshade

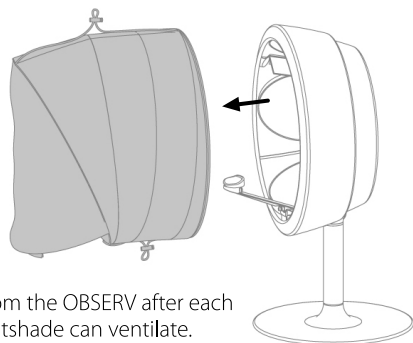




Remove the lightshade from the OBSERV after each  
use. Both OBSERV and lightshade can ventilate.

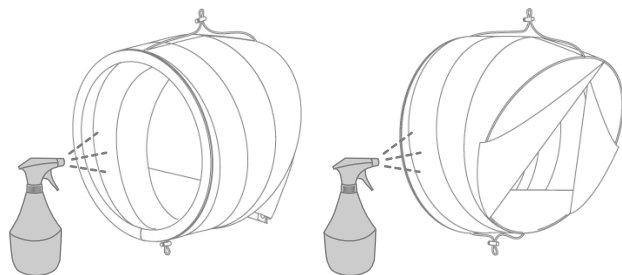

Disinfect the lightshade with medical disinfectant both inside and  
outside. Leave the lightshade to ventilate.  
Avoid pooling of liquids.

### Cleaning the device

Use medical disinfectant to clean the iPad  
and visor.

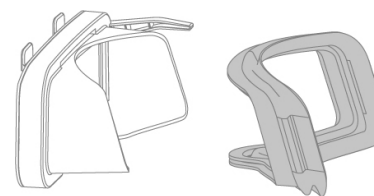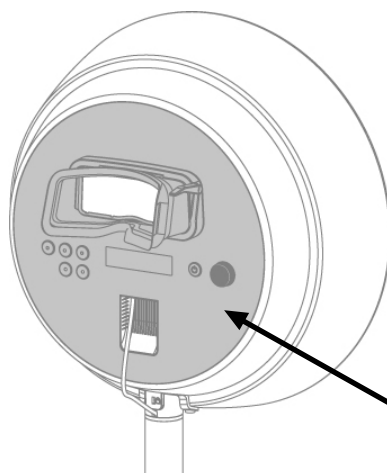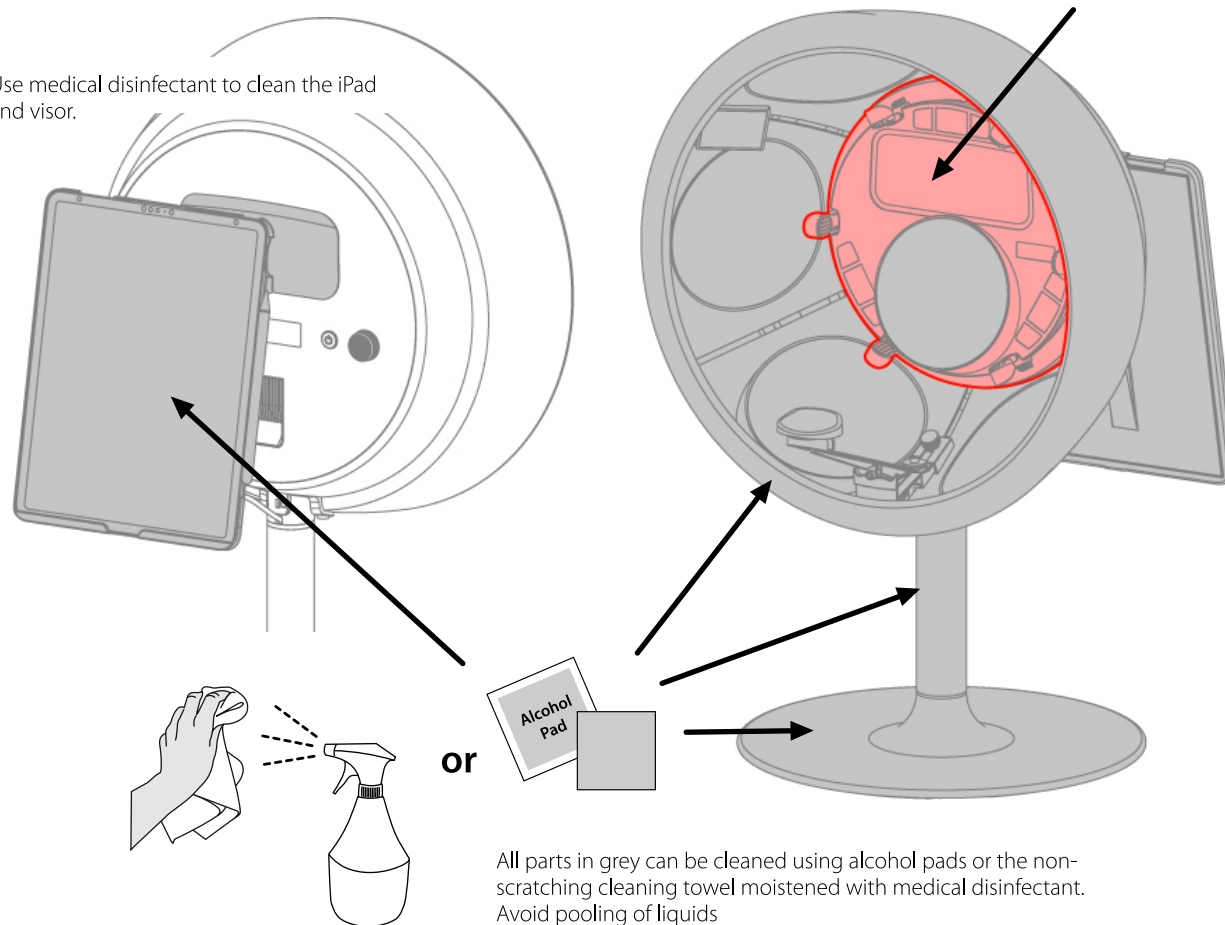

Clean the rubber of the polarisation visor after each use. To disinfect  
the rubber part remove it from the visor and use medical disinfectant.

Clean the backplate of the Observ.